




## **DETECTO IS EMR/EHR-READY!**

With increased interest in connecting Detecto's scales to Electronic Medical Records systems, it is important to know the true advantages of Detecto's USA-made power.

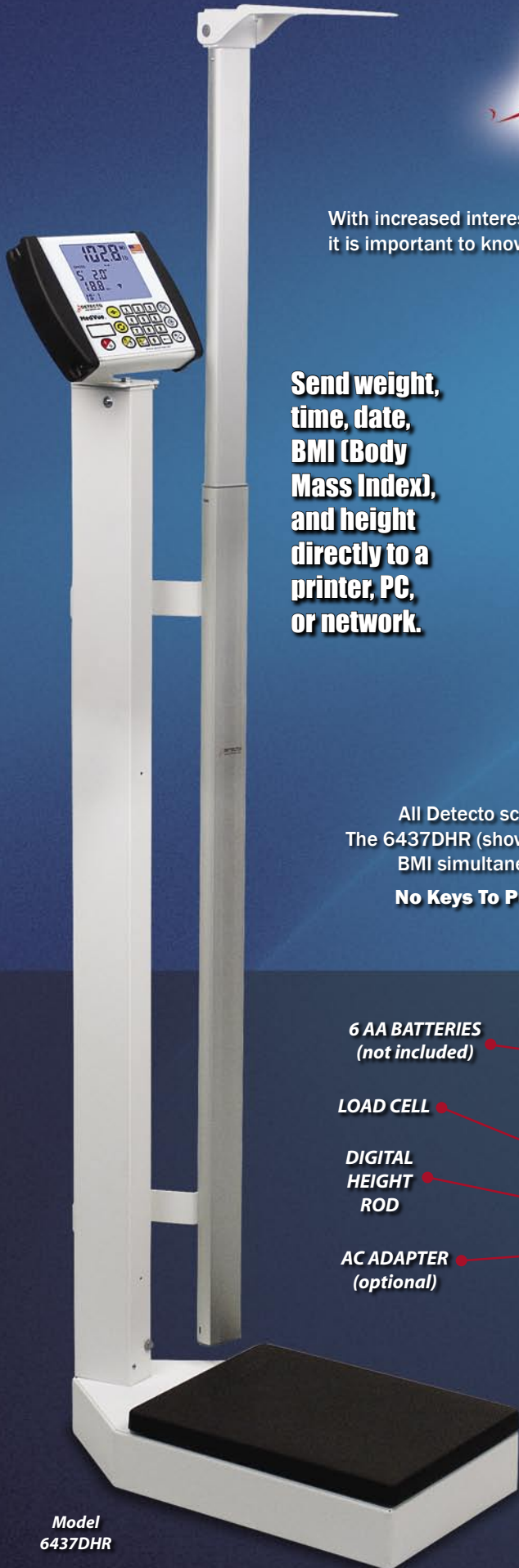

**Send weight, time, date, BMI (Body Mass Index), and height directly to a printer, PC, or network.**



*Wi-Fi  
Optional*

*MedVue  
Indicator  
Front*

## **THE POWER OF DETECTO'S MEDVUE!**

All Detecto scales with the MedVue indicator can export data to EMR/EHR software. The 6437DHR (shown to the left) is a great example. The 6437DHR displays height, weight, and BMI simultaneously by simply raising the digital height rod while weighing a patient.

**No Keys To Push! No Operational Sequences To Memorize! Fully Automatic!**




## **MEDVUE CONNECTIVITY**

All Detecto scales featuring the MedVue indicator are EMR/EHR capable. The MedVue includes a DB9 serial connector, USB port, and optional Ethernet wired or Wi-Fi card for those applications that require the scale to be on a computer network.

*Model  
6437DHR*



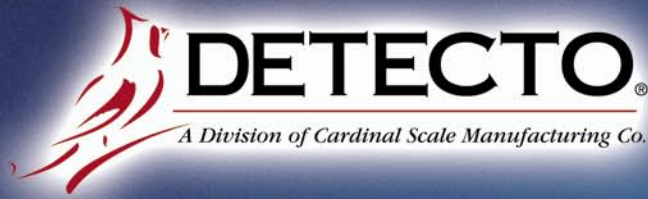

# DETECTO®

A Division of Cardinal Scale Manufacturing Co.



## TRANSMITTING DATA WITH THE MEDVUE

Detecto's MedVue supports several formats and techniques for transmitting data. Data transmission may be initiated via computer command or operator input and transmitted via RS-232, USB or optional wired or Wi-Fi Ethernet.

The simplest protocol is the P185 Journal format. When the patient is standing on the scale, adjust the digital height rod, and either press the PRINT key or invoke the Journal Print command from the connected computer. This will transmit data only when a the patient's weight/height needs to be recorded and requires very little processor time for the EMR, ensuring that the patient is on the scale with the height rod properly placed.

**The following data/information will be sent via the selected port:**

**Time, Date, ID, BMI, Height, Weight <CR><LF>**

- Time= hh:mm
- Date= mm/dd/yy
- ID= xxxxxxxxxxx (patient ID)
- BMI= xx.x (percentage)
- Height= x' xx.x" or xx.x cm (inches or centimeters)
- Weight= xxx.xuu G (gross weight)

**ASCII Output String:**

hh:mm,mm/dd/yy,xxxxxxxxxx,xx.x,x' xx.x",xxx.xuu G<CR><LF>



Model 6857DHR

*The 6857DHR scale is designed for bariatrics and nursing homes. It includes a low-profile, high-capacity scale base and handrails for patients who require support while weighing.*

## USA-MANUFACTURED DESIGN QUALITY

If a particular EMR software package requires a custom data format, Detecto Scale can provide a special quotation to output the weight record in a special format. This is all made possible since Detecto is a complete 100% software/hardware manufacturer located in Webb City, Missouri. No other medical scale company can do it like we do it... because WE MAKE IT!



*Detecto Scale reserves the right to improve, enhance, or modify features and specifications without prior notice.*

**SOLD BY:**



# DETECTO®

A Division of Cardinal Scale Manufacturing Co.

203 E. Daugherty, Webb City, MO 64870 USA

Pb: 417-673-4631 or 1-800-641-2008 Fax: 417-673-5001

[www.Detecto.com](http://www.Detecto.com)